

Bimanual High-Flow Instruments

- Lightweight titanium irrigation instruments reusable
- Lightweight colour coded irrigation instruments single-use
- Compatible with Phaco or I/A System via standard Luer connectors
- Swiss Design / Swiss Made

Irrigation high-flow tip variants



Bimanual High-Flow Irrigation Instruments reusable

Art. No.	Description
BI 121/T	Irrigation Handpiece 20G (Ø0.90mm), polished tip, dual-side ports Ø0.65mm (high flow)
BI 126/T	Irrigation Handpiece 20G (Ø0.90mm), polished tip, tip with open end 45° (high flow)
BIO 127/T	Irrigation Handpiece oval cannula (1.10x0.70mm), matt finish, tip with open end 30° (high flow)

Bimanual High-Flow I/A System single-use

Art. No.	Description
BSD 518	Bimanual I/A System single-use 21G (Ø0.80mm) set, irrigation handpiece: dual-side ports Ø0.50mm (high flow), aspiration handpiece: asp. port Ø0.35mm, rough tip

PLEASE NOTE

Instruments with oval/elliptical cannulas have a higher tightness at the cannula edge. This leads to substantial less efflux. In consequence, especially the irrigation has the potential of a substantial increase of the inner eye pressure (in particular in case of irrigation with TWIN PORT design). To prevent any damage, e.g. any form of collapse of the eye, a reduced height of the irrigation liquid bottle is strongly recommended. Keep the pressure conditions in permanent observation and, if necessary, adjust (decrease).

REUSABLE INSTRUMENTS

The expected lifetime of our reusable medical devices is validated by a defined number of processing cycles. Please refer to the information provided in the instructions for use.

