EN – INSTRUCTIONS FOR USE
Reusable Irrigation/Aspiration Systems

SCOPE AND CONTENT

These instructions for use apply to the product family I/A System reusable by Bürki inno med AG. Each package contains a non-sterile instrument (handpiece or tip). The handpieces are either the monomanual / coaxial (SL, safe-lock) system or the bimanual system (see section Monomanual (SL) System / Bimanual System).

INTENDED USE
The reusable irrigation / aspiration systems are intended to be used to deliver solutions to the eye and to extract solutions and residuals of tissue from the eye during cataract operations in the anterior chamber. The instruments are connected to the irrigation/aspiration device system by standard Luer-cones.

INTENDED USERS
The reusable irrigation/aspiration systems must only be used by suitable qualified surgeons. The surgeon is responsible for the application.

INTENDED PATIENT POPULATION
The reusable irrigation / aspiration systems are used in eye operations on the human eye in the presence of a cataract.

CONTRAINDICATIONS
No specific contraindications are known for the reusable irrigation / aspiration systems.

IMPORTANT USER INFORMATION
Insufficient flow behaviour can cause a fluid imbalance. We strongly recommend that you observe pressure ratios closely and adjust as necessary. This is essential to prevent a collapse of the anterior chamber. Bürki inno med AG refuses any liability whatsoever for the improper or incorrect handling of reusable irrigation/aspiration instruments.


MONOMANUAL (SL) SYSTEM / BIMANUAL SYSTEM
The monomanual system both irrigates solutions into the eye and extracts solutions and tissue residues from the eye in a single instrument. It can be operated with one hand. The instrument uses a standard irrigation Luer (female) and a standard aspiration Luer (male). It is connected to irrigation/aspiration tubing system via two standard Luer cones. The bimanual system irrigates the eye with solutions and extracts solutions and tissue residues from the eye via two independent handpieces. These are operated simultaneously with two hands. Each handpiece is colour-coded:

Irrigation handpiece (female Luer) **BLUE**
Aspiration handpiece (male Luer) **PURPLE**
They are connected to the irrigation/aspiration tubing system via standard Luer cones.

WARNINGS/ PRECAUTIONS (Observe packaging symbols with care.)
- Instruments must only be used by qualified, specialist personnel.
- Instruments are not sterile when delivered. Before use, they must be cleaned, disinfected and sterilized.
- The cleaning and disinfection method must follow the **CLEANING AND DISINFECTION** instructions exactly.
- The use of reusable instruments that have not been cleaned and sterilized is strictly prohibited. This can cause dangerous postoperative eye infections.
- Never use instruments if their packaging is damaged.
- Inspect tips before use. Never use a product if the tip or any other part is damaged.
- As required by good clinical practices, test for adequate irrigation and aspiration flow prior to entering the eye.
- The manufacturer bears no liability for any injury or damage suffered by a patient due to use of the product.

BIO 120/T, BIO 122/T, BIO 127/T, BIO 128/T: Instruments with oval/elliptical cannulas have a higher tightness at the cannula edge. This leads to substantial less efflux. In consequence, especially the irrigation has the potential of a substantial increase of the inner eye pressure (in particular in case of irrigation with "TWIN PORT" design: BIO 128/T). To prevent any damage, e.g. any form of collapse of the eye, a reduced height of the irrigation liquid bottle is strongly recommended. Keep the pressure conditions in permanent observation and, if necessary, adjust (decrease).

APPLICATION
- To assemble the monomanual (coaxial) I/A system (SL), the tip (B) has to be safely attached to the handle (A1) using the 'safe-lock' ring (A2): Insert the base of the tip (B) into the handle (A1). Make sure that the knob is inserted all the way into the notch of the handle's 'safe-lock' ring (A2). To lock it in place, turn the ring (A2) 180°. Then screw the irrigation silicone sleeve (C) (if tip variant is without metal sleeve) carefully onto the tip (B) and its cannula. Make sure that the aspiration cannula's port is perpendicular to the two dual-side ports of the irrigation sleeve. The end of the irrigation sleeve shall be placed 1-2 mm below the aspiration port.

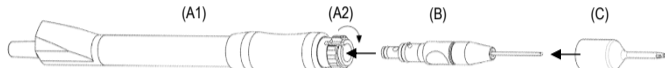- The reusable irrigation/aspiration systems are delivered non-sterile and must be treated according to the reprocessing instructions.
- The instruments are connected to the irrigation/aspiration tubing system via standard Luer connectors.
- Fill the irrigation and aspiration system with a 0.9% NaCl solution to ensure functionality.
- Also, read and follow the instructions for use of the irrigation/aspiration device system.

TECHNICAL SPECIFICATIONS

Settings	Specifications
Connectors	Standard Luer cone
Sterilization	Steam sterilization
Delivery condition	Not sterile
Product lifetime	51 processing cycles

NOTIFICATION TO MANUFACTURER AND AUTHORITIES

Serious incidents related to the product must be reported to the manufacturer and the competent authority of the country in which the user and/or patient is established.

DISPOSAL
Devices must be disposed of according to local regulations. For further information, please contact your local environmental/public authority, or a qualified waste disposal company. Waste must therefore be recycled or disposed of:

- Without harming human health
- Without using any processes or methods that are harmful to the environment, specifically water, air, soil, flora or fauna
- Without generating noise or smells

LABELLING AND SYMBOLS

	Article number		Batch code		Notified Body identification number
	Non sterile		Consult instructions for use		Medical device
	Manufacturer		Unique device identifier (01) GTIN (11) Date of manufacture (10) Batch code		Authorised representative in the European Union
	Date of manufacture		Serial number		EU importer
	Do not use if packaging is damaged		Keep dry		Keep away from sunlight
	Irrigation handle				Aspiration handle with marking "R" (right)
	Aspiration handle				Aspiration handle with marking "L" (left)
	Curved cannula (bimanual)				Curved cannula with angle (bimanual)
	Gauge size outer diameter (in G and in mm)				Aspiration cannula with angle, with port dimensions indicated
	Curved cannula (bimanual), high-flow				Flow direction
	Irrigation cannula with dual side ports, polished tip, with port diameter indicated				Irrigation cannula with dual side ports, rough tip, with port diameter indicated
	Aspiration cannula, rough tip, with port diameter indicated				Irrigation cannula with double opening ('Twin Ports'), with port diameter indicated
	Tip (monomanual, safe-lock), with metal sleeve				Handle (coaxial/ monomanual) I = irrigation A = aspiration
	Outer diameters (monomanual), in mm, outer and inner cannula				O-ring set for monomanual system
	Ports of straight cannula (monomanual), with port diameters indicated				
	Ports of curved cannula (monomanual), with port diameters indicated				
	Ports of angled cannula (monomanual), with port diameters indicated				Angled cannula (monomanual), with angle indicated
	Port of straight aspiration cannula (monomanual), with port diameter indicated				
	Port of curved aspiration cannula (monomanual), with port diameter indicated				
	Port of angled aspiration cannula with port diameter indicated				Angled aspiration cannula with angle indicated

Details of the manufacturer and the EU authorised representative

Bürki inno med AG
Industriestrasse 67
CH-9443 Widnau, Switzerland
Tel. +41 (0) 71 744 88 66
https://www.buerki.eu

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Version: P02P04-IFU-MDR (IA System reusable) 1112 EN 01 / 2026-04-17

Reprocessing instructions, according to EN ISO 17664-1
Cleaning, disinfecting and sterilizing I/A System reusable by Bürki inno med AG

PRODUCTS

These processing instructions only apply to the product family I/A System reusable (monomanual/coaxial as well as bimanual) by Bürki inno med AG. To ensure that products can be cleaned in an autoclave, please read the Instructions for use.

INTRODUCTION:

All products must be cleaned, disinfected and sterilised before every use. This also applies the first time non-sterile products are used after delivery. Proper cleaning and disinfection is an essential prerequisite to sterilizing products effectively.

WARNING BEFORE REPROCESSING:

- The user is responsible for product sterility and must ensure that:
- Only devices with the proper, validated methods are used for cleaning, disinfection and sterilization
- All equipment used (washer-disinfector, sterilizer) is maintained and checked regularly
- Validated settings are used for each cycle
- Local legal provisions and the hygiene instructions for the hospital or institution are followed

The original device packaging cannot be cleaned, disinfected or sterilized. Never use it to process the device.

CLEANING AND DISINFECTION

If possible, use an automated cleaning and disinfection method to clean and disinfect the products. Using a manual method is not recommended by the manufacturer and has not been validated. Only use a manual method if no automated method is available. In this case, please take into account the significantly lower performance and reproducibility of a manual method. If a manual cleaning and disinfection method is used, product- and process-specific validation is required, and is the responsibility of the user.

I. Pretreatment

Remove any visible contamination from the products directly after use on a patient using potable water and a soft brush (with in 15 minutes of the surgical procedure ending at most).

WARNING

- Never use ultrasound treatment as a pretreatment
- Never use fixing agents or potable water warmer than 40°C during pretreatment
- Check that the disinfectant/cleaning agent generally has been approved as effective. We recommend using e.g. *Bomix Plus* (an aldehyde-free disinfectant with cleaning properties) as a pretreatment.

The disinfectant/cleaning agent must be suitable for instruments made of metal or plastic, and be compatible with the products (see "Material resistance"). Please note that any disinfectant/cleaning agent used during pretreatment is only for the safety of personnel. In no way can it replace the disinfection carried out after cleaning.

Method

1. Disconnect the product from the irrigation/aspiration tubing system.
2. Rinse the lumen from the irrigation/aspiration system at least three times with a syringe (minimum volume 50 ml) in the direction of flow. During this process, check the patency (flow) of the instrument.
3. Remove the silicone irrigation sleeve (if present), then brush off any residue on the instrument with a soft brush.
4. Place the instrument and its silicone sleeve (if any) in the soaking bath according to the manufacturer's instructions (for the validation, *Bomix Plus* was used: the manufacturer recommends a solution of 2% for 5 minutes at maximum room temperature). Ensure the instrument and silicone sleeve are fully immersed in the disinfectant/cleaning agent and that there is no contact between instruments.
5. Remove the instrument from the pre-cleaning solution, flush the product under running potable water (at a temperature of <40°C) for at least 1 minute, then brush off any residue on the instrument with a soft brush.

WARNING

- Any cleaning agent used must be:
 - Suitable for cleaning metal or plastic instruments
 - Compatible with the products (see "Material resistance")
- Please refer to the instructions for any disinfecting/cleaning agents for information on concentration, temperature, soaking time and rinsing.

II Automated cleaning/disinfecting

The washer-disinfector must:

- Be certified for meeting performance requirements (e.g. CE marking according to EN ISO 15883 or DGHM approval)
- Feature approved programs for thermal disinfection (A0 value ≥ 3000 or, for older devices, at least 5 min. at 90°C).
- Have a suitable program for the products and include a sufficient number of flushing/rinsing steps
- Only rinse with deionized water
- Only use filtered air for drying (oil-free, minimal contamination with micro-organisms and particles)
- Be regularly maintained, inspected and calibrated
- The washer-disinfector must be fitted with flush connectors for instruments with lumen (e.g. cannulae, handpieces). Adequate, reproducible flushing pressure must be confirmed through specific validation.

WARNING

When choosing a cleaning agent, please ensure it is:

- Suitable for cleaning metal or plastic instruments
 - Compatible with the product (see the section "Materials resistance")
- You must follow the manufacturer's instructions for the cleaning agent on concentration, temperature, soaking time and rinsing. Chemical disinfection should only be used when thermal disinfection is not available. Chemical disinfection may leave dangerous residues on the product. When using chemical disinfection, product- and process-specific validation is required, under the responsibility of the user.

Method

- 1a. Connect the instruments to the appropriate connectors on the washer-disinfector cart. (SL model: with tips installed). Make sure that the instruments do not come into contact with each other and, if possible, close any unused flush connectors.
- 1b. Connect the lumen instruments (e.g. handpieces) to the appropriate connectors on the washer-disinfector cart and, if possible, close any unused flush connectors. Use the adapter provided to properly connect the instruments to the washer-disinfector.
- 1c. Place the silicone sleeves (if present) in suitable cleaning and disinfection baskets.
2. Start the program. The program sequence must include the following program steps:
 - Rinse for 1 minute with cold water at <40°C
 - Flush for 2 minutes with cold water
 - Wash for 10 minutes with 0.5% *neodisher® MediZym* at 45°C +/- 5°C
 - Flush for 1 minute with deionised water at <40°C
 - Flush for 1 minute with deionised water at <40°C
 - Thermal disinfection for 5 minutes at 92°C +/- 2°C
 - Dry for 30 minutes at 100°C (program settings)

3. After the program ends, disconnect and remove the instrument from the washer-disinfector. Instruments with lumen (e.g. cannulae, handpieces) may have to be dried further using filtered compressed air.

4. After cleaning/disinfection, all products must be checked for corrosion, damage, changes in colour and impurities for subsequent sterilization. The basic suitability of the products for effective automated cleaning and disinfection has been proven by an independent, state-accredited, recognized testing laboratory, using a Miele PG 8535 washer/disinfector, and using *neodisher® MediZym* detergent and cleaning agent (Dr. Weigert GmbH & Co. KG, Hamburg), in accordance with the stated method. If a silicone irrigation sleeve or any of the Monomanual (SL) System's 'o-rings' of the tip or the handle are damaged, replace them with new ones that have been correctly washed and disinfected.

5. Please package the cleaned, disinfected instruments in suitable sterilization packaging that meets the following requirements (material/process):

- EN ISO 11607
- Can be steam sterilized (resistant to temperatures of at least 138°C (280°F), sufficient steam permeability)
- Offer sufficient protection to the products and the sterilization packaging from mechanical damage

WARNING

Never reuse damaged products (for the maximum number of reuse cycles, see "Reusability"). Visibly contaminated products must be cleaned and disinfected again. No maintenance is required. Instrument oils must not be used.

III Sterilization

Only the following sterilization methods have been validated. Other sterilization methods must not be used.

Steam sterilization

- Method using 20 minutes of product drying and at least 3 pre-vacuum cycles.
- Steam sterilizer according to EN 13060/EN 285
- Validated according to EN ISO 17665-1
- Maximum sterilization temperature of 134°C (273°F); plus tolerance according to EN ISO 17665-1)
- Sterilization time (exposure time at sterilization temperature):

Area	Fractionated vacuum/dynamic air removal	Gravity displacement ¹
Germany	At least ² 5 minutes at 134°C (273°F)	Not recommended
Switzerland	At least ² 18 minutes at 134°C (273°F)	Not recommended
Other countries	At least ² 3 minutes at 132°C (270°F) / 134°C (273°F)	Not recommended

The fundamental suitability of the products for effective steam sterilization using the fractional vacuum/dynamic air removal method has been proven by an independent/state-accredited testing laboratory using Sterilizer EHS 3870 Tuttnauer. This took into account standard conditions in clinics and doctors' surgeries, and the specified method.

¹ The use of methods involving gravity displacement is not recommended: These require significantly longer sterilization times, and specific validation of the device, method, program, settings and products used. Any such validation is the responsibility of the user.

² Drying time required depends directly on a range of factors determined by the user (load configuration and density, sterilization conditions). Effective drying times must be determined by the user. Drying times of less than 20 minutes should not be used.

IV. Storage

After sterilization, store the products in the sterilization packaging, at room temperature and in a dry and dust-free location.

V. Materials resistance

You must ensure that the detergent/cleaning agent is an enzymatic cleaner and that the detergent/cleaning agent or disinfectant does not contain any of the substances listed below:

- Organic, mineral and oxidizing acids with pH value below 5.5
- Alkalis with pH value above 8.5 (a neutral/enzymatic cleaner is recommended)
- Organic solvents (e.g. acetone, ether, benzene)
- Oxidants (e.g. peroxide)
- Halogens (chlorine, iodine, bromine)
- Halogenated aromatic hydrocarbons

WARNING

The product does not require corrosion inhibitors, which may leave potentially dangerous residues on the product. Never use acid neutralizing agents or rinse aids, as these can leave potentially dangerous residues on the product. Never clean products with metal brushes, sharp instruments or steel wool. Never expose products to temperatures higher than 138°C.

VI. Reusability

The products can be reused up to 51 times, if appropriate care is taken during processing and if they are undamaged and clean. The user is responsible for ensuring that the products are clean and working properly after processing. The manufacturer accepts no liability for the cleanliness or functionality of the product after processing.

Monomanual system: Replace the silicone sleeves and o-rings on all reusable systems that can be dismantled after 10 cleaning/sterilization cycles.

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